

ScoutPro™

Osmolarity System

◆ QUALITY CONTROL TESTING ◆

Electronic check cards



When to test

- Test **once a day** before testing any patients.
- Test any time the pen has been dropped or mishandled.

Reminders

- Electronic check cards are blue.
- Two identical cards come in each box.
- Do **NOT** discard after use. They can be reused.
- Do **NOT** collect tears or control solutions with the electronic check cards.

How to use the electronic check cards

1. Attach electronic check card onto pen.
2. The pen will beep, and a green light will turn on.
3. A test result will appear on the display.
4. Compare the result to the expected value found in the instructions that came with the cards.
5. If results FAIL, stop testing. Contact your local rep or call technical support (855) 832-7522 and select extension 2.



QUALITY CONTROL TESTING

Control solutions



When to test

- Test with **each new shipment of test cards** that you receive (even if the lot number is the same as the previous shipment).
- Test with **each new lot number** of test cards.
- Test **each month** to check storage.

Reminders

- Control solutions are used to confirm that the test cards are working properly.
- The glass bottles (ampoules) should be kept at room temperature.
- Do not refrigerate or keep in a hot place.
- Only use the “normal” and “high” osmolarity solutions provided by Bausch + Lomb.
- Do **NOT** collect tears.

How to use the control solutions

- Use the blue sleeve to snap off the top of the ampule.
- Attach a test card to the pen.
- Within 5 seconds, enter the numeric code on top of the test card into the pen using the arrow buttons.
- Turn the ampule upside down (fluid will not spill out).
- Touch the tip of the pen to the control solution; a test result will appear on the display.
- Compare the result to the expected value found in the instructions that came with the control solutions.
- Repeat this procedure for the second control solution provided.
- If results FAIL, stop testing. Contact your local rep or call technical support (855) 832-7522 and select extension 2.

Breaking the ampule



Testing the control solution



See the ScoutPro Osmolarity System User Manual for complete safety information and instructions.

BAUSCH + LOMB
See better. Live better.