

ScoutPro™

Osmolarity System

◆ SCOUTPRO TEST PROCEDURE ◆

1



Attach test card onto the pen. The pen will beep, and a green light will turn on.

2



Within 5 seconds, enter the numeric code on the top of the test card by pressing the right (>) and left (<) buttons on the pen.

3



Securely hold the test card wings with one hand and remove the test card cover with your other hand.

4



Seat patient with head back and eyes looking up and away. Do not pull the eyelid away from the eye.

SCOUTPRO TEST PROCEDURE

5



Lower the pen until the bottom of the tip touches the line of moisture on top of the eyelid. When tears are collected, the pen will beep, and the green light will turn off.

6



A test result will appear on the display. DO NOT DOCK the pen on the charging base while waiting for results. Record the test result in the patient's chart.

7



Remove the test card by sliding forward with your index finger and discard. DO NOT reuse test cards.

8



To view previous test results, press either the right (>) or left (<) buttons on the pen.

See the ScoutPro Osmolarity System User Manual for complete safety information and instructions.

BAUSCH + LOMB
See better. Live better.